




A range of Bathroom and Shower accessories designed for use by the young and elderly and by those with limited mobility.



Hinged Safety Grab Rail with adjustable Leg (1 of 2 pages)

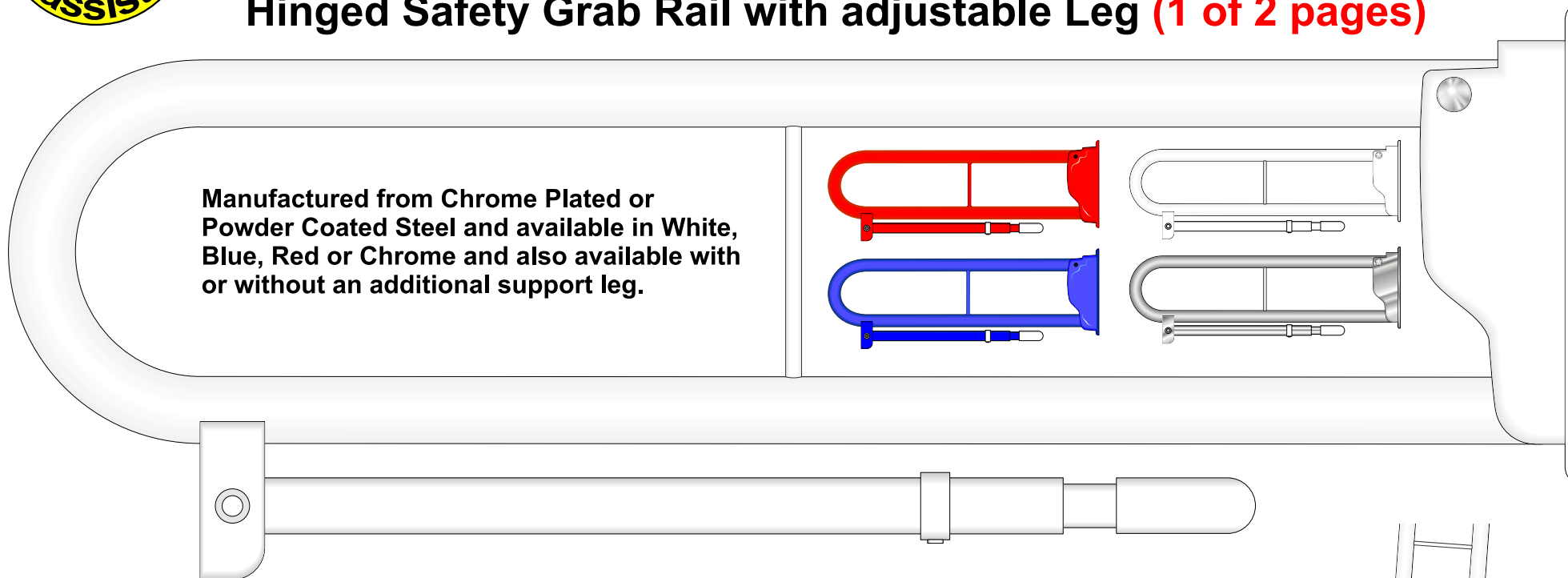

Manufactured from Chrome Plated or Powder Coated Steel and available in White, Blue, Red or Chrome and also available with or without an additional support leg.


Instructions Please note that fixings are not included with this device as the fixing method will depend on the structure onto which the rail is to be secured. If fitting to a concrete or masonry wall, make sure that good quality plugs and screws are used.

Do not fix onto a plasterboard wall unless backing timbers or suitable alternative of sufficient strength are used to ensure a safe and adequate assembly.



If the support leg version is used, note that a simple to adjust clip is fitted which enables two variations in the length of the leg to be chosen.

Incorporates a simple "lift and lock" safety mechanism which keeps the Grab Rail upright against the wall when not in use.

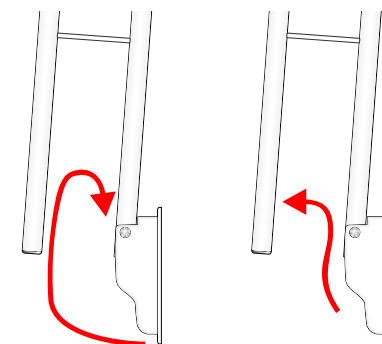




A range of Bathroom and Shower accessories designed for use by the young and elderly and by those with limited mobility.



Hinged Safety Grab Rail with adjustable Leg (2 of 2 pages)

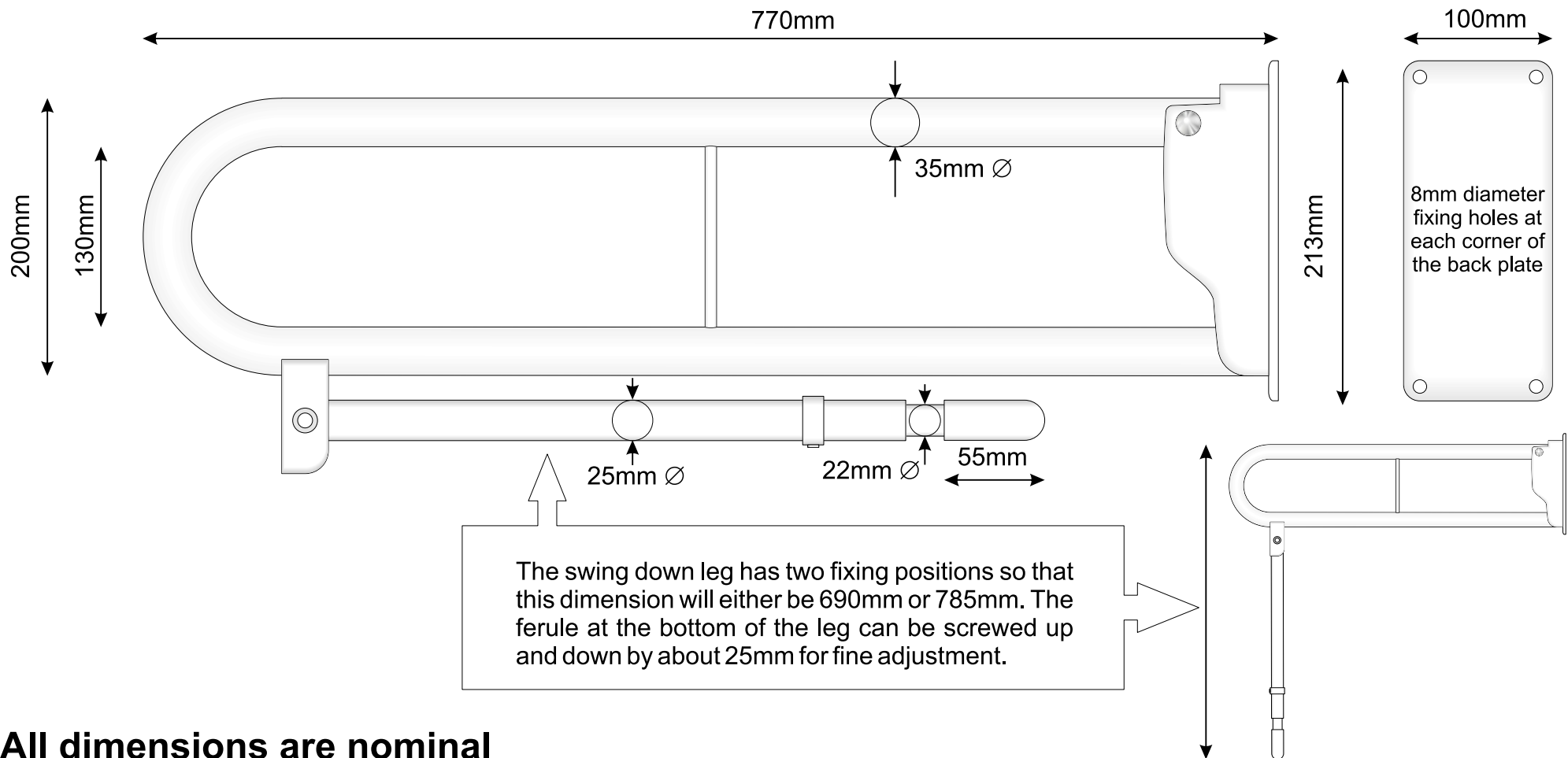

All dimensions are nominal